



Anya

User Manual

IMPORTANT:
Do not use without having first read this manual.

Information before using the product / the functioning of medical products

Dear purchasers, we congratulate you for purchasing the HealthCo product. Please take the following points into consideration:

1. Please read the enclosed instructions for use carefully.
2. Use the product only for the intended purpose of use.
3. Product management should be allowed only for the personnel familiar with the functioning of the product.
4. The maintenance as well as the safety and technical control must be carried out only by HealthCo or by any other professional company authorised by HealthCo.
5. Assembly, extensions, revisions or repairs must be carried out only by HealthCo or by any other professional company authorised by HealthCo.

In case of any technical interventions in the sense of additions or revisions as regards the products that are not carried out by HealthCo or any other professional company authorised by HealthCo, any guarantee in relation to revisions as well as in relation to the product and its functions referring to such revision, shall be abolished.

Preventive measures

Please read the instructions for use carefully. Use the product only for the intended purpose.

The remarks under the heading »WARNING« must be taken into consideration.

Purpose of use

The trolley is designed to allow the care-receiver to stand up in a fast and stable manner and to overcome short distances. It is also intended for transfer of care-receivers, who can stand, but have difficulties moving. The Anya trolley allows the transfer of a care-receiver between the bed, the wheelchair, and the toilet, etc. The trolley can be fixed by means of two separate brakes located on the rear wheels or by means of a central brake. Two front rotating wheels are intended for ease of manoeuvring. The trolley is also equipped with a safety belt. Padded knee support provides support to the care-receiver. The trolley consists of two main parts (a handle and a base), which allows simple assembly and disassembly.

Regulations and norms

Provided that certain norms or regulations are valid, they should be taken into account.

CE mark

CE-mark refers to the guideline 93/42/EWG for medicinal products.

Responsibility and guarantee

The manufacturer of the product is responsible for the safety and reliability of the product only if:

- A. the product is used solely in the sense of the purpose of use.
- B. the controls over the functions are carried out on a regular basis.

Transport damages

Upon delivery, please check if the product or the packaging is damaged in any way. In this relation, the declaration date should be taken into consideration: there is a 2-day deadline for visible damages and a 7-day deadline for hidden damages.

Serial number

The type designation of the HealthCo Anya can be found on the lower frame. Please state the serial number at enquiries.

Product display

1. Handles / three heights
2. Knee suport
3. Plate for standing
4. Wheels with break 2X
5. Wheels without break 3X



Technical information

Width:	558 mm
Length:	670 mm
Height:	1180 mm
Weight without packaging:	16 kg
SWL:	150 kg

Product management

The Anya is ready to be used right after removing the packaging.

Use of Anya:

Accommodation of the patient step by step as shown:









WARNING

When accommodating the patient, the central brake must be braked.



Maintenance

The product should be wiped with a duster. Approximately every two months the framework should be polished with a polishing paste. Knee and thigh supports are made of a material that is easy to maintain – composition leather, tested according to DIN 54020. Such coating may be wiped with a dry or damp cloth; detergent for sensitive laundry can also be used if necessary but it should be wiped straight after it has been used (the stains should be removed as soon as possible in order to prevent the absorption in the material). Finally, the material should be cleaned with warm and clean water.

Ingrained dirt can be removed if rubbed with a cloth, moistened with alcohol (the cloth should not be wet). Cleaning agents that contain fats or oils should not be used. Similarly, solvents (like acetone) or rough cleaning agents should not be used.

Safety and technical control

In accordance with »Medicinal Products Act« the safety and technical control should be carried out once a year.

HOW TO USE ANYA WITH SAFETY BELT

Anti-slip sliding belt will help you by the lifting of the patient.



Belt is used to help lift and transport the patient.

First place patients legs on the plastic plate.

Then place the belt around the lumbar part of the patient



Before using, belt needs to be fixed on the belt bracket on one side of the Anya.



With one hand hold the belt and with the other pull the belt. Belt will slide and patient will slowly stand up on the Anya.

Next step is to hang the belt to the other side belt bracket. This will protect the patient from falling.