

HealthCo
LTD



Isla User Manual

IMPORTANT:
Do not use without having first read this manual.

w: healthcoltd.co.uk

Information before using the product / the functioning of medical products

Dear purchasers, we congratulate you for purchasing the HealthCo product. Please take the following points into consideration:

1. Please read the enclosed instructions for use carefully.
2. Use the product only for the intended purpose of use.
3. Product management should be allowed only for the personnel familiar with the functioning of the product.
4. The maintenance as well as the safety and technical control must be carried out only by HealthCo or by any other professional company authorised by HealthCo.
5. Assembly, extensions, revisions or repairs must be carried out only by HealthCo or by any other professional company authorised by HealthCo.

In case of any technical interventions in the sense of additions or revisions as regards the products that are not carried out by HealthCo or any other professional company authorised by HealthCo, any guarantee in relation to revisions as well as in relation to the product and its functions referring to such revision, shall be abolished.

Preventive measures

Please read the instructions for use carefully. Use the product only for the intended purpose.

The remarks under the heading »WARNING« must be taken into consideration.

Purpose of use

This Mover is used for helping the patients when standing up from the bed, toilet, chair or wheelchair and for transporting of the patient. It is intended for patients who can stand, but not necessarily walk.

Regulations and norms

Provided that certain norms or regulations are valid, they should be taken into account.

CE mark

CE-mark refers to the guideline 93/42/EWG for medicinal products.

Responsibility and guarantee

The manufacturer of the product is responsible for the safety and reliability of the product only if:

- A. the product is used solely in the sense of the purpose of use.
- B. the controls over the functions are carried out on a regular basis.

Transport damages

Upon delivery, please check if the product or the packaging is damaged in any way. In this relation, the declaration date should be taken into consideration: there is a 2-day deadline for visible damages and a 7-day deadline for hidden damages.

Serial number

The type designation of the HealthCo Eva + HealthCo Isla can be found on the lower frame. Please state the serial number at enquiries.

Product display – HealthCo Isla

1. Handles
2. Handle for adjustable base
3. Tight support
4. Knee support
5. Wheels with break
6. Plate stand
7. Wheels without break



Adjustable base



- a.) MIDDLE – closed
- b.) LEFT POSITION – inside from 45 cm to 75,50 cm
- c.) RIGHT POSITION – outside from 57 cm to 91,50 cm

Technical information

Width: 640 mm

Length: 930 mm

Height: 1050 mm

**Flexible base: from 58 cm to 97,0 cm – outside
from 45 cm to 83,00 – inside**

Weight without packaging: 33 kg

SWL: 150 kg

w: healthcoltd.co.uk

t: 01709 278036 | e: enquiries@healthcoltd.co.uk


The Croft House, York Lane, Rotherham, South Yorkshire, S66 9JH

Product management

The Mover is ready to be used right after removing the packaging.

Use of Mover:

Open both tight supports by pulling them with hand on the side and upwards.

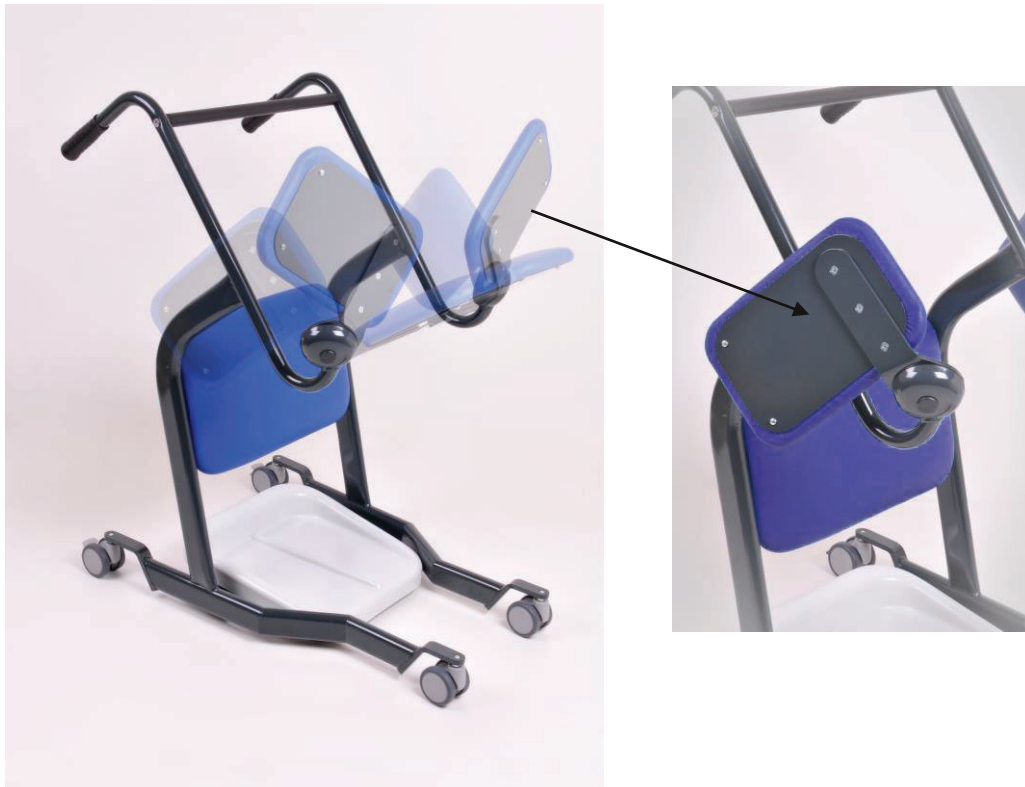

Accommodation of the patient step by step as shown:



WARNING

When accommodating the patient the wheels must be braked.



Maintenance

The product should be wiped with a duster. Approximately every two months the framework should be polished with a polishing paste. Knee and thigh supports are made of a material that is easy to maintain – composition leather, tested according to DIN 54020. Such coating may be wiped with a dry or damp cloth; detergent for sensitive laundry can also be used if necessary but it should be wiped straight after it has been used (the stains should be removed as soon as possible in order to prevent the absorption in the material). Finally, the material should be cleaned with warm and clean water.

Ingrained dirt can be removed if rubbed with a cloth, moistened with alcohol (the cloth should not be wet). Cleaning agents that contain fats or oils should not be used. Similarly, solvents (like acetone) or rough cleaning agents should not be used.

Safety and technical control

In accordance with »Medicinal Products Act« the safety and technical control should be carried out once a year.