



Selina Bariatric User Manual

IMPORTANT:
Do not use without having first read this manual.

Information before using the product / the functioning of medicinal products

Dear purchasers,

we congratulate you for purchasing the HealthCo product. Please take the following points into consideration:

1. Please read the enclosed instructions for use carefully.
2. Use the product only for the intended purpose of use.
3. Product management should be allowed only for the personnel familiar with the functioning of the product.
4. The maintenance as well as the safety and technical control must be carried out only by HealthCo or by any other professional company authorised by HealthCo.
5. Assembly, extensions, revisions or repairs must be carried out only by HealthCo or by any other professional company authorised by HealthCo.

In case of any technical interventions in the sense of additions or revisions as regards the products that are not carried out by HealthCo or any other professional company authorised by HealthCo, any guarantee in relation to revisions as well as in relation to the product and its functions referring to such revision, shall be abolished.

Preventive measures

Please read the instructions for use carefully. Use the product only for the intended purpose.

The remarks under the heading »WARNING« must be taken into consideration.

Purpose of use

This walking-assistant trolley is used for supporting the patients while walking.

WARNING: The Koval walking-assistant trolley does not have any brakes. Therefore it must not be used by those patients who are incapable of stopping the trolley with their own strength.

Regulations and norms

Provided that certain norms or regulations are valid, they should be taken into account.

CE mark

CE-mark refers to the guideline 93/42/EWG for medicinal products.

Responsibility and guarantee

The manufacturer of the product is responsible for the safety and reliability of the product only if:

- A. the product is used solely in the sense of the purpose of use.
- B. the controls over the functions are carried out on a regular basis.

Transport damages

Upon delivery, please check if the product or the packaging is damaged in any way. In this relation, the declaration date should be taken into consideration: there is a 2-day deadline for visible damages and a 7-day deadline for hidden damages.

Serial number

The type designation of the Selina walking-assistant trolley can be found on the lower frame. Please state the serial number at enquiries.

Explanation of marks

MPG	Medicinal Products Act
93/42/EWG	a guideline for medicinal products
CE	CE mark

Use only in inner premises.

Product display

Figure 1:

The display of the trigger for height adjustment



Figure 2:

The display of the walking-assistant trolley with handles

Adjustable handles

Screw for width adjustment

Attachable levers for handles

Gas spring with reinforcement tube

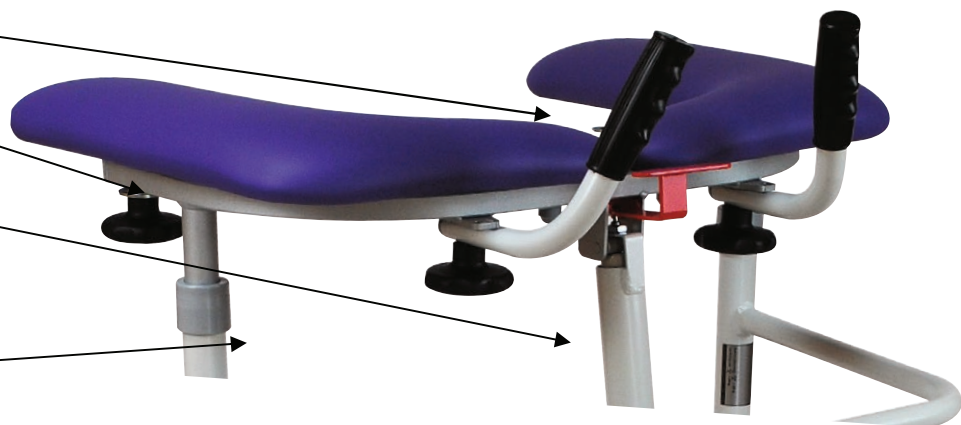


Figure 3:

Shock absorber and sliding rails



Figure 4:


Bottom side attachment of the gas spring

Wheels 100 mm with brakes



Technical information

Width:	max. 88 cm
Length:	max. 75 cm
Height adjustment:	104-134 cm
Weight without packaging:	ca. 34 kg
Lift:	2 x shock absorber 200 N
Max. patient weight:	250 kg
SWL:	300 kg

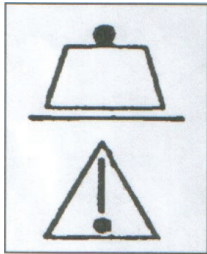

Product management

The walking-assistant trolley is ready to be used right after removing the packaging.

Height adjustment with the aid of the built in shock absorber

The upper part of the walking-assistant trolley can be adjusted by means of the trigger lever (Figure 1).

Lifting the upper part: push the handle downwards; due to the shock absorber pressure, the uplift is enabled.

Lowering the upper part: push the handle downwards; due to the slight pressure, the upper part is now ready to be pushed downwards.

Please note that drag can occur in the shock absorber if the latter remains unused for a longer period of time. In this way, it can be brought to a halt.

WARNING

After the shock absorber has been left unused for a long time, a slight pressure in the shock absorber can be eliminated if the shock absorber is pushed downwards.

Troubleshooting

The height adjustment of the trolley is impossible:

POSSIBLE CAUSES:

SOLUTION:

The adjustment of the shock absorber is impossible. There is drag in the shock absorber.

Push the shock absorber downwards.

The shock absorber is blocked.

Loosen the lower attachment of the shock absorber and turn it around in a clockwise direction. Reattach the shock absorber.

The trolley refuses to move.

The rear wheels are blocked due to the brakes. Loosen the brakes.

Repair, servicing and spare parts

In case of any technical problems, please contact: