


40038-40048

HealthCo Supreme Fit Deluxe, Loop Sling 40050-40060

HealthCo Supreme Fit Deluxe w/ Headsupport, Loop Sling

EN Instruction for use
FR Instructions d'utilisation
ES Instrucciones de uso
GB Gebrauchsanweisung
IT Istruzioni per l'uso
NL Instructies voor gebruik
CN 使用說明書

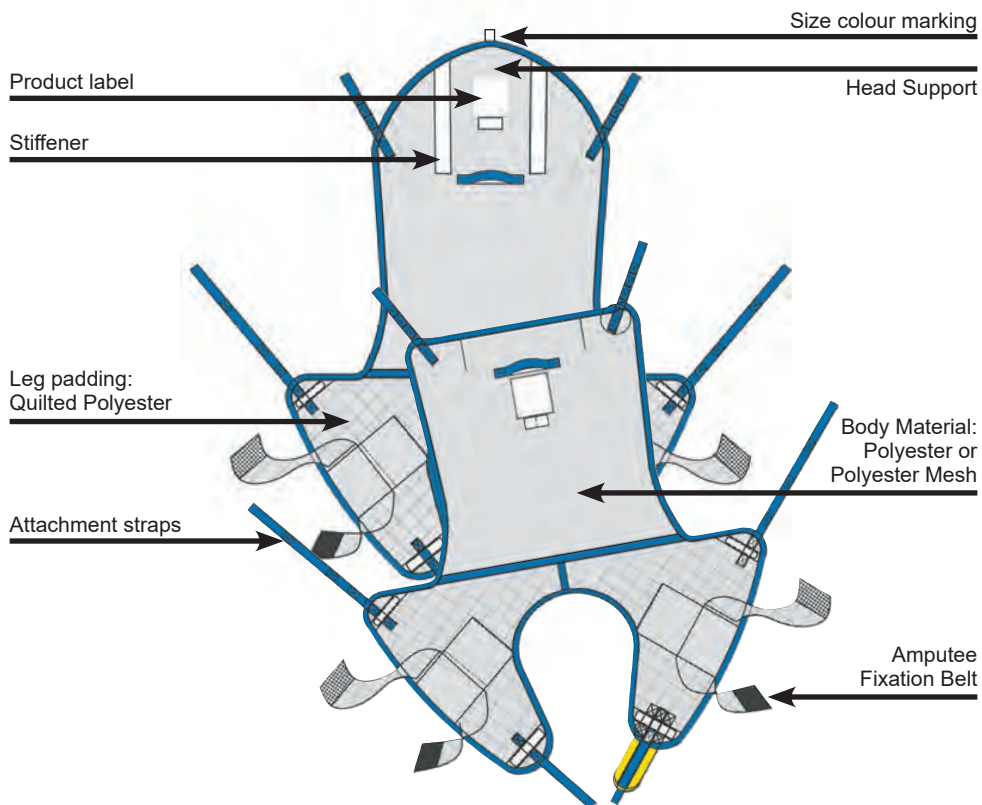

Washing instructions:



Warning! - Avertissement! - Advertencia! - Attenzione! - Warnung! - Waarschuwing - 警告! 請

Always read this instruction and the instruction for the lifter to be used with the sling
Toujours lire cette instruction et l'instruction pour l'élévateur à être utilisés avec la fronde
Siempre lea esta Instrucción y la Instrucción para el elevador que se utilizarán con la Honda
Lesen Sie immer diese Anleitung und die Anleitung für die Heber mit der Schlinge verwendet werden
Leggere sempre questa Istruzione e l'Istruzione per il sollevatore da utilizzare con la fionda
Lees altijd deze instructie en de instructie voor de lifter te gebruiken met de draagdoek
务必阅读本使用说明书以及使用吊兜的升降机的说明书。

Technical Specifications (Lifting Slings are classified as medical equipment Class 1 products.)

Part no.	Description	SWL	Material	Sizes
40038-40048	Supreme Fit Deluxe, Loop Sling	Max: 275kg	Polyester	XSmall-XXLarge
40050-40060	Supreme Fit Deluxe w/ HS, Loop Sling	/ 605 lbs.		

Definition in this IFU

Warning: Means Safety warning. Failure to understand or obey this warning may result in injury to you or to others.

Note: Means important information for the correct use of this system or equipment.

Intended Use

The HealthCo Supreme Fit Deluxe sling is designed with split legs and shaped body support for users with good head control.

Suitable for most transfers in a seating or laying position for users with good head control.

The Supreme Fit Deluxe HS sling (with Head Support) is designed with split legs and full shaped body for users who requires head support. Suitable for most transfers in a seating or laying position and where user have little neck and/or head control.

The sling is for use with lifters/hoists and hanger bars with loop attachment hooks. The Supreme Fit Deluxe Sling will assist the caregiver for lifting and transfer a pa-tient in either a sitting or laying position, from and to the bed and/ or wheelchair.

Safety instructions

- Always check the label is readable and that the sling is in good condition without signs of wear and tear.
- Always make sure not to exceed the Safe Working Load (SWL) defined for the product.
- Always read the instruction manual for the lifting equipment to be used with the sling.

Before and after every use

1. Always perform a risk assessment before starting the specific lifting session in order to make sure the product model is correct and that the lifting procedure is fully understood.
2. Before using the sling, read this instruction manual and the instruction manual for the lifting equipment to be used
3. When soiled or stained the sling must be washed before use.
4. **Warning:** Check the sling for any fraying, loose stitching, attachments or any deteriorations.
5. After use, check sling according to the step 3 and 4 above and store in a dry area.

Attachments

The Supreme Fit Deluxe Loop Sling will fit most lifts with standard hangerbars with loop attachment hooks.

Always check lifter instruction manual for details.

Service & Maintenance

The sling shall be checked by authorised personnel every 6 months according to ISO10535

Customer support/Trouble shooting

- Please refer to the manual for the lifting equipment used
- In doubt please contact your dealer who supplied the product

Expected service life & Disposal

Expected Service Life varies between 1–5 years, depending on usage and number of washing cycles. When washed once a week the sling has an Expected Service Life of 2 years. Expected Service Life is based upon a correct usage and washings procedure in accordance with this IFU. Dispose the slings according to the local regulations.

Storing and Shelf life

The sling should be stored in a dry and clean environment. The shelf life is 3 years in original packaging. Store in temperatures between 0°-40° C (+32°-104°F)

Safe Working Load (SWL)

Max 275kg / 605 lbs.


Product care & Maintenance

- Always follow the instructions on the product label and check the sling for deteriorations.
- Mark the sling with the specific patient name and keep in dry and clean environment.


Applying the Supreme Fit Deluxe Sling in Chair

1. Lean the user slightly forward and position the Sling behind the users back and head. The label side facing outward and the center line of the sling aligned with the user's spine. Carefully push the sling as far down the patient lower back as possible. (See picture 1)

Note! If used on a leg amputated user, attach fixation belt on the amputee side and secure it around the residual limb.

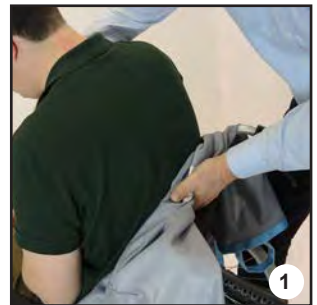


Place the sling legs alongside the user's legs. Fold the sling legs and carefully feed it underneath the user's legs. (See picture 2,3,4)

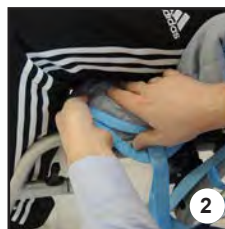

After positioning the sling legs cross the short yellow strap and put both thigh straps through it. Then cross each thigh strap with it's own leg strap. (See picture 5,6)



Note! If required, two flat slide sheets can be used to apply the sling. (See picture 7,8)


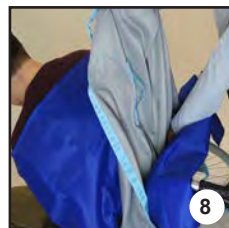


2. Recline the user into the sling, ensuring the arms are contained within the sling.

Warning! Make sure that the sling wont get caught in the armrest or anywhere else. (See picture 9,10)



3. Position the Hanger bar carefully directly above the user. (See picture 11,12)

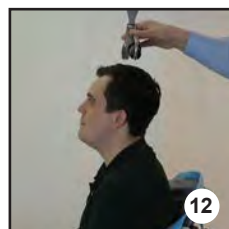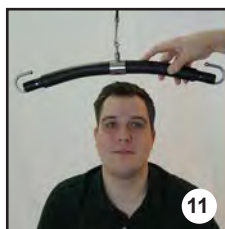

4. Place the four loop straps on the hanger bar attachments hooks and pull the strap down until locked in position. (See picture 13,14)

Note: Always check the lifter instruction manual details.



5. Check user position and carefully make the required adjustments to achieve the best comfort for the user.

Check the support of the head and that arms and legs are in the right positions.
(See picture 15)



6. Raise the hanger bar until all straps start to tighten and become tense enabling you to check **ALL** the straps are securely attached.
(See picture 16)



7. Carefully perform the lift, whilst remaining vigilant of the user's position and comfort.
(See picture 17)




8. Once the process of transfer is complete, return the user to a chair or bed and slowly lower the hanger bar until all straps are loose.
(See picture 18)



Release the loop straps or clips from the hanger bar by pulling/pushing the strap upwards.
(See picture 19,20)

Note: Always check the lifter instruction manual details.

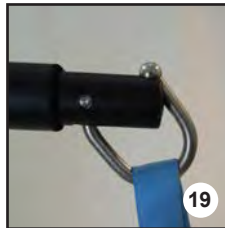

Applying the Supreme Fit Deluxe Sling in Laying position

1. Roll the user to the side, facing away from you.

Roll up top half or the sling and gently tuck it under the user.
(See picture 1)




Fit the sling by folding it on the middle.
The label side facing outwards.
Then place the sling lengthwise with the users back. (See picture 2,3,4)




2. Roll user to the other side, now facing you.
Gently pull the top half of the sling through, when fully unraveled flatten it.
Roll the user to a backside position.
(See picture 5,6)

Note! If used on a leg amputated user, attach fixation belt on the amputee side and secure it around the residual limb.

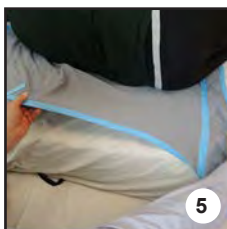

3. Insert sling legs under the user thigh.
(See picture 7,8)

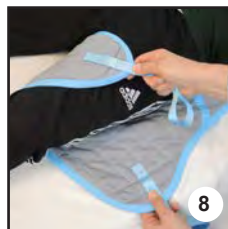

After positioning the sling legs cross the short yellow strap and put both thigh straps through it.
Then cross each thigh strap with its own leg strap.
(See picture 9,10,11)



Note! If required, two flat slide sheets can be used to apply the sling.
(See picture 12,13)



4. Position the hanger bar carefully above the user.
(See picture 14,15)



5. Place the four loops straps on the hanger bar attachment hooks and pull the straps down until locked in position.
(See picture 16,17)

Note: Always check the lifter instruction manual details.




6. Check user position and carefully make the required adjustments to achieve the best comfort for the user. Check the support of the head and that arms and legs are in the right positions.
(See picture 18)

Note: If required, recline the bed so the user is in a half-sitting position.



7. Raise the hanger bar until all straps start to tighten and become tense, enabling you to check that **ALL** the straps are securely attached."

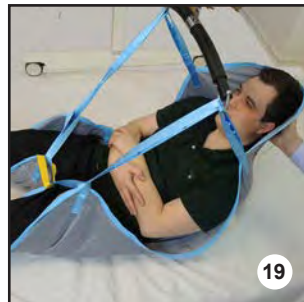

8. Carefully perform the lift, whilst remaining vigilant of the user's position and comfort. Remember to support the user's head when raising from the bed.
(See picture 19,20)



9. Once the process of transfer is complete, return the user to a chair or bed and slowly lower the hanger bar until all straps are loose.
(See picture 21)



10. Release the loop strap from the hanger bar by pulling the strap upwards.
(See picture 22,23)

Note: Always check the lifter instruction manual details.



Description of label symbols. Applicable symbols printed on product label
Description des symboles de l'étiquette. Symboles applicables imprimés sur l'étiquette du produit
Descripción de los símbolos de la etiqueta. Símbolos aplicables impresos en la etiqueta del producto
Beschreibung der Label-Symbole. Anwendbar Symbole auf Produktetikett
Descrizione dei simboli di etichette. Simboli applicabili stampati sull'etichetta del prodotto
Beschrijving van het etiket symbolen. Toepasselijke symbolen afgedrukt op etiket

标签符号描述，印在产品标签上适用的符号。



Patient Name
 Le nom du patient
 Nombre del paciente
 Patientenname
 Nome del paziente
 Naam van de patiënt
 患者姓名



Date of manufacturer / month
 Date de fabrication / mois
 Fecha de fabricación / mes
 Herstellungsdatum / Monat
 Data di produzione / mese
 Fabricagedatum / maand
 製造日期



Do Not Bleach
 Ne pas javelliser
 No usa blanqueador
 Nicht bleichen
 Non candeggiare
 Niet bleken
 不能漂白。



Manufacturer
 Fabricant
 Fabricante
 Hersteller
 Fabricante
 Fabrikant
 生產廠家



Read the instruction for use
 Lire les instructions d'utilisation
 Lea las instrucciones de uso
 Lesen Sie die Gebrauchsanweisung
 Leggere le istruzioni per l'uso
 Lees de instructies voor gebruik
 請閱讀使用說明書



Do Not Tumble Dry
 Ne pas sécher par culbutage
 No utilizar secadora
 Nicht im Wäschetrockner trocknen
 Non asciugare
 Niet in de wasdroger
 不可烘乾



Article number
 Numéro de l'article
 Número del artículo
 Artikel-Nummer
 Numero di articolo
 Artikelnummer
 貨號



EU representative
 EU Représentant
 EU Representante
 EU Vertreter
 EU Rappresentante
 EU Vertegenwoordiger
 EU 代表



Tumble Dry at Low Heat
 Sécher par culbutage à basse température
 Secar a baja Heat
 Wäschetrockner trocknen bei niedriger Hitze
 Nell'asciugatrice a bassa calore
 Drogen op lage temperatuur
 低温滚筒烘干



Lot Serial number
 Numéro de série de LOT
 Número de serie LOT
 LOT Seriennummer
 Numero di serie LOT
 LOT serienummer
 批号



Declaration of compliance
 Déclaration de conformité
 Declaración de cumplimiento
 Konformitätserklärung
 Dichiarazione di conformità
 Verklaring van conformiteit
 符合性聲明



Do Not Iron
 Ne pas repasser
 No planchar
 Nicht bügeln
 Non stirare
 Niet strijken
 不可熨燙



Serial Number
 Numéro de série
 Número de serie
 Volnummer
 Numero di serie
 Volnummer
 序列号



Attachment by straps with loops
 Fixation par sangles avec boucles
 Fijación por correas con bucles
 Befestigung durch Gurte mit Schlaufen
 Fissaggio con cinghie con anelli
 Bevestiging door bandjes met lussen
 通过吊带连接



Do Not Dry Clean
 Ne pas nettoyer à sec
 No lavar en seco
 Nicht chemisch reinigen
 Non lavare a secco
 Nicht chemisch reinigen
 不可乾洗



Material Symbol
 Symbole du matériel
 Símbolo material
 Materiaien symbol
 Simbolo material
 Materiaal symbol
 材質符號



Attachment by straps with Clips
 Fixation par sangles avec clips
 Fijación por correas con clips
 Befestigung durch Gurte mit clips
 Fissaggio con cinghie con clips
 Bevestiging door bandjes met clips
 通过扣子连拉



Recommended temperature for washing
 La température recommandée pour le lavage
 La temperatura recomendada para el lavado
 Empfohlene Temperatur zum Waschen
 Temperatura consigliata per il lavaggio
 Aanbevolen temperatuur voor het wassen
 推薦使用溫度洗滌



Service label
 Étiquette de service
 Servicio de etiqueta
 Service-Etikett
 Etichetta di servizio
 Dienst label
 服務標籤



Safe Working Load
 Capacité de levage maximum
 Carga de trabajo segura
 Maximale hebelast
 Carico massimo di sollevamento
 Maximale tilcapaciteit
 安全工作載荷



Do not wash
 Ne pas laver
 No Lave
 Non lavare
 Nicht waschen
 Niet wassen
 不可水洗



Medical Device
 Dispositif medical
 Producto sanitario
 Medizimprodukt
 Dispositivo medico
 Medisch hulpmiddel
 醫療器械



Do not use.
 Ne pas utiliser
 No utilize
 nicht verwenden
 Non usare
 Gebruik geen
 不要使用



HealthCo Limited
 The Croft House, York Lane
 Morthen, South Yorkshire S66 9JH, UK
 Tel. +44(0)1709 278036
 www.healthcoldt.co.uk